

# ImmunoGuide®

## Instructions for Use

### Vedolizumab ELISA (mAb-based)

Enzyme immunoassay for the specific and quantitative determination of free Vedolizumab in serum and plasma

REF: IG-AB116



12X8



2-8°C



**AYBAYTECH** Biyoteknoloji Ltd. Sti.  
Macun Mah. Batı Blv. ATB İş Merkezi No: 1/285,  
06374-Yenimahalle, Turkey  
Tel : +90 312 397 88 05  
Fax : +90 312 397 88 06  
e-mail: [info@aybaytech.com](mailto:info@aybaytech.com)  
[www.aybaytech.com](http://www.aybaytech.com)

<b><u>Contents</u></b>	<b><u>Page</u></b>
1. Intended Use	2
2. Summary and Explanation	2
3. Principle of the Test	2
4. Warnings and Precautions	2
5. Storage and Stability of the Kit	3
6. Specimen Collection, Handling and Storage	3
7. Contents of the Kit	4
8. Materials Required but not Supplied	4
9. Procedure Notes	5
10. Pre-Test Setup Instructions	5
10.1. Preparation of Components	5
10.2. Dilution of Standards and Samples	6
11. Test Procedure	6
11.1. General Remarks	6
11.2. Assay Procedure	7
11.3. Quality Control	8
11.4. Calculation of Results	8
12. Assay characteristics	9
12.1. Specificity	9
12.2. Sensitivity	10
12.3. Precision	10
12.4. Recovery	10
13. Automation	10
14. References	10

### 1. INTENDED USE

Enzyme immunoassay for the specific and quantitative determination of free Vedolizumab in serum and plasma.

### 2. SUMMARY AND EXPLANATION

The drug Vedolizumab (trade name Entyvio<sup>®</sup>) is a humanised immunoglobulin G1 monoclonal antibody that binds exclusively to the lymphocyte integrin  $\alpha 4\beta 7$ , is indicated for the treatment of adult patients with moderately to severely active ulcerative colitis [UC] or Crohn's disease [CD]. According to the prescribing information, the mean  $\pm$  SD of Vedolizumab steady-state trough serum concentrations in patients with UC and CD are reported to be  $11.2\pm 7.2$   $\mu\text{g/mL}$  ( $n=77$ ) and  $13.0\pm 9.1$   $\mu\text{g/mL}$  ( $n=72$ ) respectively. It was also reported in another study that higher Vedolizumab trough concentrations were associated with greater efficacy in patients with both UC and CD. Higher Vedolizumab trough concentrations also were associated with numerically higher rates of clinical response and mucosal healing at week 6 in patients with UC. As with all therapeutic proteins, there is a potential for immunogenicity. According to the prescribing information, in patients who received Vedolizumab, the frequency of antibodies detected in patients was 13% at 24 weeks after the last dose of study drug. During treatment, 56 of 1434 (4%) of patients treated with Vedolizumab had detectable anti-Vedolizumab antibodies at any time during the 52 weeks of continuous treatment. Nine of 56 patients were persistently positive for anti-Vedolizumab antibodies and 33 of 56 patients developed neutralizing antibodies to Vedolizumab. The presence of persistent anti-Vedolizumab antibodies was observed to substantially reduce serum concentrations of Vedolizumab, either to undetectable or negligible levels at weeks 6 and 52.

Identification of biomarkers might be beneficial for (non-) response and risk factors for adverse drug reactions that might be related to serum drug levels and maintaining the effective concentration in order to potentially avoid some side effects with a reliable method.

### 3. PRINCIPLE OF THE TEST

This ELISA is based on a Vedolizumab-specific monoclonal antibody (catcher Ab, *ImmunoGuide* clone19F3). Diluted standards and samples are incubated in the microtiter plate coated with IG-19F3 mAb. After incubation, the wells are washed. A horseradish peroxidase (HRP)-conjugated anti-human IgG antibody is added and binds to the Fc part of Vedolizumab. Following incubation, wells are washed, and the bound enzymatic activity is detected by addition of chromogen-substrate. The color developed is proportional to the amount of Vedolizumab in the sample or standard. Results of samples can be determined by using the standard curve.

#### 4. WARNINGS AND PRECAUTIONS

1. Before starting the assay, read the instructions completely and carefully. Use the valid version of the package insert provided with the kit. Be sure that everything is understood. For further information (clinical background, test performance, automation protocols, alternative applications, literature, etc.) please refer to the local distributor.
2. In case of severe damage of the kit package, please contact **AybayTech** or your supplier in writing, latest one week after receiving the kit. Do not use damaged components in test runs, but keep safe for complaint related issues.
3. Obey lot number and expiry date. Do not mix reagents of different lots. Do not use expired reagents.
4. Follow good laboratory practice and safety guidelines. Wear lab coats, disposable latex gloves and protective glasses where necessary.
5. Reagents of this kit containing hazardous material may cause eye and skin irritations. See MATERIALS SUPPLIED and labels for details.
6. Chemicals and prepared or used reagents have to be treated as hazardous waste according the national biohazard safety guidelines or regulations.
7. Avoid contact with Stop solution. It may cause skin irritations and burns.
8. If any component of this kit contains human serum or plasma it is indicated and if so, it have been tested and were found to be negative for HIV I/II, HBsAg and HCV. However, the presence of these or other infectious agents cannot be excluded absolutely and therefore reagents should be treated as potential biohazards in use and for disposal.
9. Some reagents contain preservatives. In case of contact with eyes or skin, flush immediately with water.

#### 5. STORAGE AND STABILITY OF THE KIT

The kit is shipped at ambient temperature and should be stored at 2-8°C. Keep away from heat or direct sun light. The storage and stability of specimen and prepared reagents is stated in the corresponding chapters. The microtiter strips are stable up to the expiry date of the kit in the broken, but tightly closed bag when stored at 2-8°C.

#### 6. SPECIMEN COLLECTION, HANDLING AND STORAGE Serum, Plasma (EDTA, Heparin)

The usual precautions for venipuncture should be observed. It is important to preserve the chemical integrity of a blood specimen from the moment it is collected until it is assayed. Do not use grossly hemolytic, icteric or grossly lipemic specimens. Samples appearing turbid should be centrifuged before testing to remove any particulate material.

Storage:	2-8°C	≤-20°C (Aliquots)	Keep away from heat or direct sun light Avoid repeated freeze-thaw cycles
Stability:	3 d	6 mon	

## 7. CONTENTS OF THE KIT

QUANTITY	COMPONENT
1 x 12 x 8	<b>Microtiter ELISA Plate</b> Break apart strips coated with anti-Vedolizumab monoclonal antibody.
5 x 0.5 mL	<b>Vedolizumab Standards A-E, Concentrate (10X)</b> 2000; 600; 200; 60; and 0 ng/mL Used for construction of the standard curve. Contains Vedolizumab, proteins, preservative and stabilizer.
1 x 12 mL	<b>Assay Buffer</b> Blue colored. Ready to use. Contains proteins and preservative.
1 x 60 mL	<b>Dilution Buffer, Concentrate (5X)</b> Orange colored. Contains proteins and preservative.
1 x 12 mL	<b>Enzyme Conjugate</b> Red colored. Ready to use. Contains horseradish peroxidase(HRP)-conjugated anti-human IgG antibody, Proclin® and stabilizers.
1 x 12 mL	<b>TMB Substrate Solution</b> Ready to use. Contains 3,3',5,5'-Tetramethylbenzidine (TMB).
1 x 12 mL	<b>Stop Solution</b> Ready to use. 1 N Hydrochloric acid (HCl).
1 x 50 mL	<b>Wash Buffer, Concentrate (20x)</b> Contains buffer, Tween® 20 and Kathon™.
2 x 1	<b>Adhesive Seal</b> For sealing microtiter plate during incubation.

## 8. MATERIALS REQUIRED BUT NOT SUPPLIED

1. Micropipettes (< 3% CV) and tips to deliver 5-1000 µL.
2. Bidistilled or deionised water and calibrated glasswares (e.g. flasks or cylinders).
3. Wash bottle, automated or semi-automated microtiter plate washing system.
4. Microtiter plate reader capable of reading absorbance at 450 nm (reference wavelength at 600-650 nm is optional).
5. Absorbent paper towels, standard laboratory glass or plastic vials, and a timer.

## 9. PROCEDURE NOTES

1. Any improper handling of samples or modification of the test procedure may influence the results. The indicated pipetting volumes, incubation times, temperatures and pre-treatment steps have to be performed strictly according to the instructions. Use calibrated pipettes and devices only.
2. Once the test has been started, all steps should be completed without interruption. Make sure that required reagents, materials and devices are prepared readily at the appropriate time. Allow all reagents and specimens to reach room temperature (20-25 °C) and gently swirl each vial of liquid reagent and sample before use. Mix reagents without foaming.
3. Avoid contamination of reagents, pipettes and wells/tubes. Use new disposable plastic pipette tips for each reagent, standard or specimen. Do not interchange the caps of vials. Always cap not used vials. Do not reuse wells or reagents.
4. Use a pipetting scheme to verify an appropriate plate layout.
5. Incubation time affects results. All wells should be handled in the same order and time sequences. It is recommended to use an 8-channel Micropipettor for pipetting of solutions in all wells.
6. Microplate washing is important. Improperly washed wells will give erroneous results. It is recommended to use a multichannel pipette or an automatic microplate washing system. Do not allow the wells to dry between incubations. Do not scratch coated wells during rinsing and aspiration. Rinse and fill all reagents with care. While rinsing, check that all wells are filled precisely with Wash Buffer, and that there are no residues in the wells.
7. Humidity affects the coated wells. Do not open the pouch until it reaches room temperature. Unused wells should be returned immediately to the resealed pouch including the desiccant.

## 10. PRE-TEST SETUP INSTRUCTIONS

### 10.1. Preparation of Components\*


Dilute/dissolve	Component		Diluent	Relation	Remarks	Storage	Stability
10 mL	Wash Buffer	up to 200 mL	Distilled Water	1:20	Warm up at 37°C to dissolve crystals. Mix vigorously.	2-8 °C	4 w
10 mL	Dilution Buffer	up to 50 mL	Distilled Water	1:5		2-8 °C	4 w

\* Prepare Wash and Dilution Buffers before starting the assay procedure.

## 10.2. Dilution of Standards and Samples

The dilutions depicted below are examples of how to obtain final dilutions for standards and samples. Standards and samples should be properly diluted as homogenous mixture before starting the assay procedure. It is recommended mixing the standards and samples well to homogenous solution at each dilution step.

1. 20  $\mu\text{L}$  of standard added to 180  $\mu\text{L}$  of 1X dilution buffer, giving the total volume of 200  $\mu\text{L}$  and a dilution of 1:10.
2. 10  $\mu\text{L}$  of sample added to 90  $\mu\text{L}$  of 1X dilution buffer, giving the total volume of 100  $\mu\text{L}$  and a dilution of 1:10.
3. 10  $\mu\text{L}$  of 1:10 diluted sample added to 490  $\mu\text{L}$  of 1X dilution buffer, giving the total volume of 500  $\mu\text{L}$  and a dilution of 1:500. This second dilution should not be done with the standards.
4. Samples with a drug concentration above the measuring range should be rated as ">highest standard". The result should not be extrapolated. The sample in question should be further diluted with 1X Dilution Buffer and then retested.

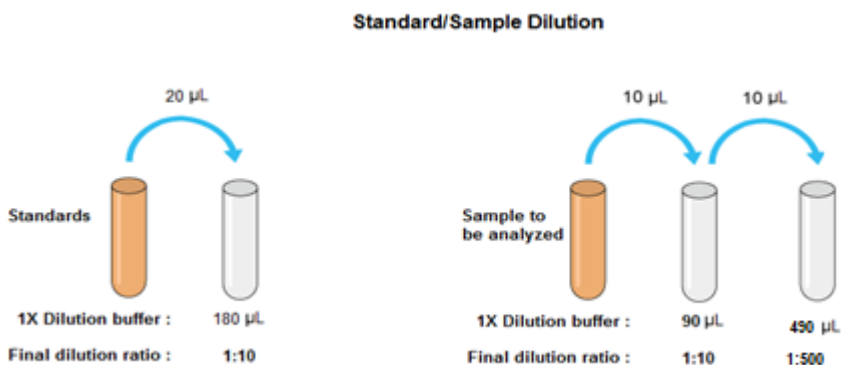

## 11. TEST PROCEDURE

### 11.1. GENERAL REMARKS

11.1.1. Before performing the assay, samples and assay kit should be brought to room temperature (about 30 minutes beforehand) and ensure the homogeneity of the solution.

11.1.2. All Standards should be run with each series of unknown samples.

11.1.3. Standards should be subject to the same manipulations and incubation times as the samples being tested.

11.1.4. All steps of the test should be completed without interruption.

11.1.5. Use new disposable plastic pipette tips for each reagent, standard or specimen in order to avoid cross contamination.

11.1.6. The total pipetting time needed for dispensing all samples into the wells should not exceed 5 minutes. If this is difficult to achieve the samples should be pre-dispensed in a separate neutral polypropylene microplate and then transferred into the reaction ELISA plate by a multi channel pipette.

## 11.2. ASSAY PROCEDURE

1.	Pipette <b>100 µl</b> of <b>Assay Buffer</b> into each of the wells to be used.
2.	<p>Pipette <b>50 µL</b> of each <b>1:10 Diluted Standard, and 1:500 Diluted Samples</b> into the respective wells of the microtiter plate. Bubble formation during the pipetting of standards and samples must be avoided.</p> <p><u>Wells</u></p> <p>A1: Standard A            B1: Standard B            C1: Standard C            D1: Standard D            E1: Standard E            F1 and so on: Samples (Serum/Plasma)</p>
3.	Cover the plate with adhesive seal. Shake plate carefully by tapping several times. <b>Incubate the plate on bench top for 60 min at room temperature (RT, 20-25°C)</b> without shaking.
4.	Remove adhesive seal. Aspirate or decant the incubation solution. Wash the plate <b>5 X 350 µL</b> of <b>Diluted Wash Buffer</b> per well. Remove excess solution by tapping the inverted plate on a paper towel.
5.	Pipette <b>100 µL</b> of <b>Enzyme Conjugate</b> (HRP-anti human IgG) into each well.
6.	Cover plate with adhesive seal. Shake plate carefully by tapping several times. <b>Incubate the plate on a bench top for 30 min</b> at RT without shaking.
7.	Remove adhesive seal. Aspirate or decant the incubation solution. Wash the plate <b>5 X 350 µL</b> of <b>Diluted Wash Buffer</b> per well. Remove excess solution by tapping the inverted plate on a paper towel.
8.	Pipette <b>100 µL</b> of Ready-to-Use <b>TMB</b> Substrate Solution into each well.
9.	<b>Incubate 15 min</b> at RT. Avoid exposure to direct sunlight.
10.	Stop the substrate reaction by adding <b>100 µL</b> of <b>Stop Solution</b> into each well. Color changes from blue to yellow. Briefly mix contents by gently shaking the plate.
11.	Measure optical density (OD) with a photometer at <b>450 nm</b> (Reference at OD620 nm is optional) within <b>15 min</b> after pipetting the Stop Solution.



### 11. 3. QUALITY CONTROL

The test results are only valid if the test has been performed following the instructions. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable standards/laws. All standards/controls must be found within the acceptable ranges as stated above and/or label. If the criteria are not met, the run is not valid and should be repeated. In case of any deviation, the following technical issues should be reviewed: Expiration dates of (prepared) reagents, storage conditions, pipettes, devices, incubation conditions and washing methods.

### 11. 4. CALCULATION OF RESULTS

A standard curve should be calculated using the standard concentration (X-axis) versus the OD450 (or OD450/620) values (Y-axis). This can be done manually using graph paper or with a computer program. Concerning the data regression by computer, it is recommended to primarily use the "4 Parameter Logistic (4PL)" or alternatively the "point-to-point calculation". In case of manual plot there are 2 options: Semilog graph (see Fig. A) or linear graph (see Fig. B). Semilog graph paper is available at <http://www.papersnake.com/logarithmic/semilogarithmic/>.


The concentration of the samples can be read from this standard curve as follows. Using the absorbance value for each sample, determine the corresponding concentration of the drug from the standard curve. This value always has to be multiplied by the individual dilution factor (usually 500). If any diluted sample is reading greater than the highest standard, it should be further diluted appropriately with 1X Dilution Buffer and retested. Also this second dilution has to be used for calculation the final result.

**It is proposed that a sample with an OD equal or lower than the standard D should NOT be plotted on the standard curve, instead should be reported as "below the detection limit".**

## Typical Calibration Curve

(All steps were performed at 23°C. Just an example. Do not use it for calculation!)

1:10 Diluted Standard	A	B	C	D	E
Concentration (ng/mL)	200	60	20	6	0
Mean OD <sub>450/620 nm</sub>	2.624	1.132	0.464	0.185	0.087

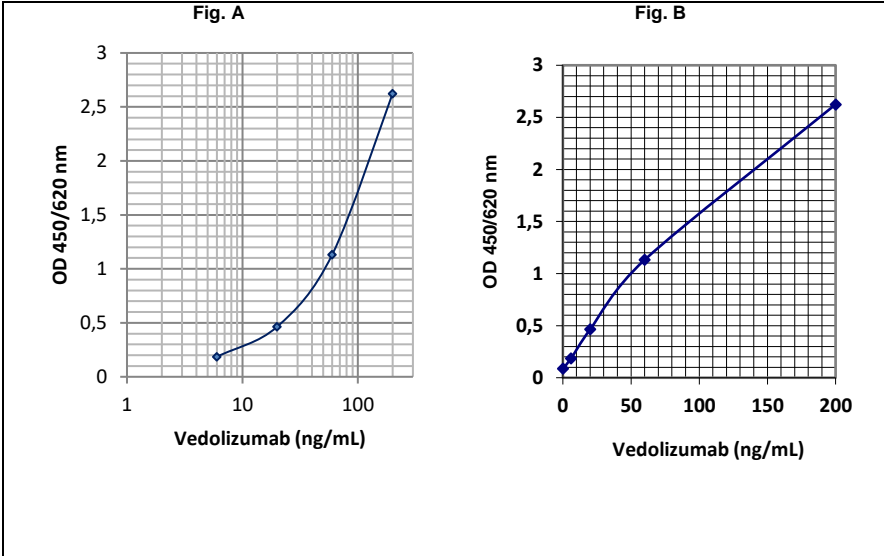

## 12. ASSAY CHARACTERISTICS

### 12.1. SPECIFICITY

There is no cross reaction with any other proteins present in native human serum. A screening test was performed with 24 different native human sera. All produced OD<sub>450/620 nm</sub> values (ranged from 0.097 to 0.166) less than the mean OD (0.311) of standard D (6 ng/mL). In addition, binding of Vedolizumab to the solid phase is inhibited by recombinant human  $\alpha 4\beta 7$  integrin protein in a concentration dependent manner. Therefore, the *ImmunoGuide* Vedolizumab ELISA (mAb-based) measures the biologically active free form of Vedolizumab, i.e. not pre-occupied by human  $\alpha 4\beta 7$  integrin antigen. No cross reaction was observed with sera spiked with the other therapeutic antibodies including Infliximab, Adalimumab, Etanercept, Bevacizumab, Cetuximab, Tocilizumab, Rituximab, and Nivolumab at concentrations up to 100  $\mu\text{g/mL}$ . All produced mean OD<sub>450/620 nm</sub> values (ranged from 0.035 to 0.058) less than standard D.

## 12.2. SENSITIVITY

The lowest detectable level that can be clearly distinguished from the zero standard is 5 ng/mL mL (zero standard +2SD read from the curve) under the above-described conditions. Analytical sensitivity is 5 ng/mL, and corresponding to the detection limit (limit of quantification) of 2.5 µg/mL for undiluted clinical samples because the serum or plasma samples are instructed to be diluted at 1:500 before starting the assay.

## 12.3. PRECISION

**Intra-assay CV:** <10%.

**Inter-assay CV:** <10%.

## 12.4. RECOVERY

Recovery rate was found to be >95% with native serum and plasma samples when spiked with exogenous Vedolizumab.

## 13. AUTOMATION

The *ImmunoGuide* Vedolizumab ELISA (mAb-based) is suitable also for being used by an automated ELISA processor.

## 14. REFERENCES

1. Williet N, Boschetti G, Fovet M, Di Bernado T, Claudez P, Del Tedesco E, Jarlot C, Rinaldi L, Berger A, Phelip JM, Flourie B, Nancey S, Paul S, Roblin X., Association Between Low Through Levels of Vedolizumab During Induction Therapy for Inflammatory Bowel Diseases and Need for Additional Doses Within 6 Months., *Clin Gastroenterol Hepatol*. 2016; S1542-3565(16)31121-1. doi: 10.1016/j.cgh.2016.11.023.
2. Wyant T, Leach T, Sankoh S, Wang Y, Paolino J, Pasetti MF, et al., Vedolizumab affects antibody responses to immunisation selectively in the gastrointestinal tract: randomised controlled trial results., *Gut*. 2014;64(1):77-83.
3. Wang MC, Zhang LY, Han W, Shao Y, Chen M, Ni R, Wang GN, Wei FX, Zhang YW, Xu XD, Zhang YC., PRISMA--efficacy and safety of vedolizumab for inflammatory bowel diseases: a systematic review and meta-analysis of randomized controlled trials., *Medicine (Baltimore)*. 2014; 93(28):e326. doi: 10.1097/MD.0000000000000326.
4. Poole RM., Vedolizumab: first global approval. *Drugs*. 2014;74(11):1293-303.
5. Milch C, Wyant T, Xu J, Parikh A, Kent W, Fox I, Berger J, Vedolizumab, a monoclonal antibody to the gut homing α4β7 integrin, does not affect cerebrospinal fluid T-lymphocyte immunophenotype, *Journal of Neuroimmunology*, 2013; 264(1–2): 123–126.
6. Bruce E. Sands, Brian G. Feagan, Paul Rutgeerts, Jean-Frédéric Colombel, William J. Sandborn, Richmond Sy, Geert D'Haens et al., Effects of Vedolizumab Induction Therapy for Patients With Crohn's Disease in Whom Tumor Necrosis Factor Antagonist Treatment Failed, *Gastroenterology* 2014;147:618–627.
7. Soler D, Chapman T, Yang LL, Wyant T, Egan R, Fedyk ER., The binding specificity and selective antagonism of vedolizumab, an anti-alpha4beta7 integrin therapeutic antibody in development for inflammatory bowel diseases., *J Pharmacol Exp Ther*. 2009;330(3):864-75.
8. Rosario M, French JL, Dirks NL, Sankoh S, Parikh A, Yang H, Danese S, Colombel JF, Smyth M, Sandborn WJ, Feagan BG, Reinisch W, Sands BE, Sans M, Fox I. Exposure-Efficacy Relationships for Vedolizumab Induction Therapy in Patients with Ulcerative Colitis

- or Crohn's Disease. *Crohns Colitis*. 2017 Feb 17. doi: 10.1093/ecco-jcc/jjx021. [Epub ahead of print]
9. Sands BE, Sandborn WJ, Van Assche G, Lukas M, Xu J, James A, Abhyankar B, Lasch K. Vedolizumab as Induction and Maintenance Therapy for Crohn's Disease in Patients Naïve to or Who Have Failed Tumor Necrosis Factor Antagonist Therapy. *Inflamm Bowel Dis*. 2017;23(1):97-106.
  10. Cohen LB, Nanau RM, Delzor F, Neuman MG. Biologic therapies in inflammatory bowel disease., *Transl Res*. 2014;163(6):533-56.
  11. Rosario M, Wyant T, Leach T, Sankoh S, Scholz C, Parikh A, Fox I, Feagan BG. Vedolizumab Pharmacokinetics, Pharmacodynamics, Safety, and Tolerability Following Administration of a Single, Ascending, Intravenous Dose to Healthy Volunteers. *Clin Drug Investig*. 2016;36(11):913-923.
  12. Amiot A, Grimaud JC, Peyrin-Biroulet L, Filippi J, Pariente B, et al. Observatory on Efficacy and of Vedolizumab in Patients With Inflammatory Bowel Disease Study Group.; Groupe d'Etude Therapeutique des Affections Inflammatoires du tube Digestif.. Effectiveness and Safety of Vedolizumab Induction Therapy for Patients With Inflammatory Bowel Disease. *Clin Gastroenterol Hepatol*. 2016;14(11):1593-1601.
  13. Loftus EV Jr, Colombel JF, Feagan BG, Vermeire S, Sandborn WJ, Sands BE, Danese S, D'Haens GR, Kaser A, Panaccione R, Rubin DT, Shafran I, McAuliffe M, Kaviya A, Sankoh S, Mody R, Abhyankar B, Smyth M. Long-term Efficacy of Vedolizumab for Ulcerative Colitis. *J Crohns Colitis*. 2016 Sep 28. pii: jjw177. [Epub ahead of print].
  14. Dulai PS, Singh S, Jiang X, Peerani F, Narula N, Chaudrey K, Whitehead D, Hudesman D, Lukin D, Swaminath A, Schmidt E, Wang S, Boland BS, Chang JT, Kane S, Siegel CA, Loftus EV, Sandborn WJ, Sands BE, Colombel JF. The Real-World Effectiveness and Safety of Vedolizumab for Moderate-Severe Crohn's Disease: Results From the US VICTORY Consortium. *Am J Gastroenterol*. 2016;111(8):1147-55.
  15. Rosario M, Dirks NL, Gastonguay MR, Fasanmade AA, Wyant T, Parikh A, Sandborn WJ, Feagan BG, Reinisch W, Fox I. Population pharmacokinetics-pharmacodynamics of vedolizumab in patients with ulcerative colitis and Crohn's disease. *Aliment Pharmacol Ther*. 2015;42(2):188-202.
  16. Colombel JF, Sands BE, Rutgeerts P, Sandborn W, Danese S, D'Haens G, Panaccione R, Loftus EV Jr, Sankoh S, Fox I, Parikh A, Milch C, Abhyankar B, Feagan BG. The safety of vedolizumab for ulcerative colitis and Crohn's disease. *Gut*. 2016 Feb 18. pii: gutjnl-2015-311079. doi: 10.1136/gutjnl-2015-311079. (Epub ahead of print)
  17. Sandborn WJ, Feagan BG, Rutgeerts P, Hanauer S, Colombel JF, Sands BE, Lukas M, Fedorak RN, Lee S, Bressler B, Fox I, Rosario M, Sankoh S, et al.; GEMINI 2 Study Group. Vedolizumab as induction and maintenance therapy for Crohn's disease. *N Engl J Med*. 2013;369(8):711-21.
  18. Fedyk ER, Wyant T, Yang LL, Csizmadia V, Burke K, Yang H, Kadambi VJ. Exclusive antagonism of the  $\alpha 4 \beta 7$  integrin by vedolizumab confirms the gut-selectivity of this pathway in primates. *Inflamm Bowel Dis*. 2012;18(11):2107-19.
  19. Feagan BG, Greenberg GR, Wild G, Fedorak RN, Paré P, McDonald JW, Cohen A, Bitton A, Baker J, Dubé R, Landau SB, Vandervoort MK, Parikh A. Treatment of active Crohn's disease with MLN0002, a humanized antibody to the alpha4beta7 integrin. *Clin Gastroenterol Hepatol*. 2008;6(12):1370-7.
  20. Yajnik V, Khan N, Dubinsky M, Axler J, James A, Abhyankar B, Lasch K. Efficacy and Safety of Vedolizumab in Ulcerative Colitis and Crohn's Disease Patients Stratified by Age. *Adv Ther*. 2017;34(2):542-559.